

We're specialists in fixation products.

Find out more about **Optimum Medical** products, including our **Ugo Fix** range, by visiting:

[www.optimummedical.co.uk](http://www.optimummedical.co.uk)



**optimum**medical

**Optimum Medical**

First floor, Unit 5, Armley Court, Armley Road,  
Leeds, LS12 2LB, United Kingdom

Tel.: +44 (0) 8456 435 479, Fax: +44 (0) 1142 383 826

[www.optimummedical.co.uk](http://www.optimummedical.co.uk)



Company Registration No: 6779883 OMS-MFG-200 Oct16

**multifix**<sup>TM</sup>  
gentle

universal  
securement device  
for medical tubings


with **silicone technology**  
for atraumatic patient experience



**optimum**medical®



universal securement device  
for medical tubings

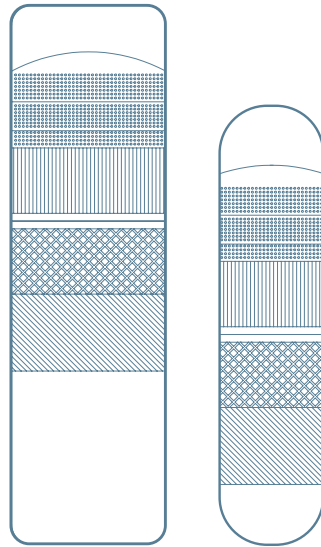

large



small



up to 7 days

## Secure, reliable and gentle.

The **Multi Fix Gentle** is a universal securement device for medical tubings and urinary catheters. It has been developed to provide comfortable fixation, giving the patient the confidence to move around freely.

**Multi Fix Gentle** is available in 2 sizes to suit a range of applications, and each one can be used for up to 7 days.

## 'Proper securement is a priority'<sup>1</sup>

The **Multi Fix Gentle** benefits from **soft silicone technology** to ensure it's easy to apply, reposition and there is minimal trauma when it's removed.

## SOFT SILICONE RE-ADHESIVE GEL STRIP

## Because '...Some patients suffer from impaired skin integrity that makes stabilization more challenging'<sup>2</sup>


**Multi Fix Gentle** has been designed to conform to the skin with a soft silicone re-adhesive gel strip which provides a range of patient-focused benefits including:

- Atraumatic removal from the skin without use of solutions to dissolve adhesive
- No sticky residue left behind
- Conforms to all skin types through multiple points of contact
- Can be repeatedly applied to same area of skin
- Re-adhesive tack enables easy repositioning
- No complex skin preparation required (no need to shave hairy skin before application)
- Reduced risk of allergic reactions due to silicone technology
- Showerproof, breathable and translucent

## OFFERING EFFECTIVE TUBE FIXATION

Suitable for the fixation  
of a range of medical tubings

- Able to remove and reposition tubing
- Leaves no residue on tubings
- Can accommodate range of tubing diameters

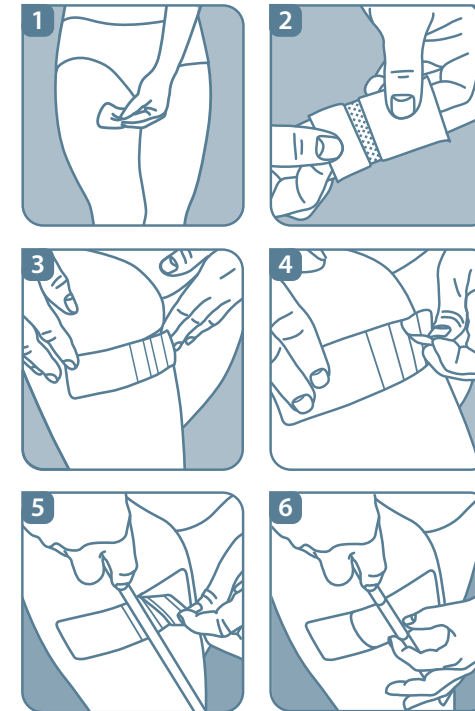

## INSTRUCTIONS FOR USE

- 1 • Ensure the skin is clean and dry (there is no need to prepare the skin or shave)
- 2 • Remove the backing from the **Multi Fix Gentle**
- 3 • Place the **Multi Fix Gentle** onto the skin in the desired spot and smooth in place
- 4 • Lift the front tab on the **Multi Fix Gentle** strip and remove the release paper to reveal the adhesive
- 5 • Place the catheter or tube on the base pad and replace the adhesive tab back in place over the catheter or tubing to secure
- 6 • Ensure your catheter or tubing is not pulling and you are able to move around comfortably with it in place.

Should you need to reposition the **MultiFix Gentle** simply peel off and replace on the skin. Additionally, the tubing can also be repositioned as necessary.

OM Code	Product Description	Size	Inner Pack	Outer Pack
3030	<b>Multi Fix Gentle</b> (large)	140X40mm	Box of 10	Case of 200
3031	<b>Multi Fix Gentle</b> (small)	110X25mm	Box of 10	Case of 200

## REFERENCES

- <sup>1</sup> Bell M A (2010) Severe Indwelling Urinary Catheter-Associated Urethral Erosion in four elderly men. *Ostomy and Wound Management*. December 2010 pp36-39
- <sup>2</sup> Callan L. & Parker D. et al. (2012) Wound, Ostomy and Continence Nurse Society, Clinical Practice Continence Committee. *Indwelling Urinary Catheter Securement: Best Practice for Clinicians*; Available at <http://www.faet.org/docs/indwellingurinarycathetersecurement.pdf>